Rev. 0 - 08/09\_108101

| Material          |
|-------------------|
| Polyamid (PA)     |
| Polyurethan (PUR) |
| Viskose (CV)      |
| Polyester (PES)   |
| Elastodien (ED)   |
| Elastan (EL)      |

| Material           |
|--------------------|
| Polyamide (PA)     |
| Polyurethane (PUR) |
| Rayon (CV)         |
| Polyester (PES)    |
| Elastodiene (ED)   |
| Spandex (EL)       |

**Bauerfeind AG**  
 Weissendorfer Strasse 5  
 07937 Zeulenroda-Triebes  
 Germany  
 Phone +49 (0) 36628-66-40 00  
 Fax +49 (0) 36628-66-44 99  
 E-mail [info@bauerfeind.com](mailto:info@bauerfeind.com)

## MalleoTrain® S

Aktivbandage  
 Active support  
 Orthèse active  
 Vendaje activo

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Stand der Information: August 2009

### GB Dear customer,

Thank you for choosing a product from Bauerfeind. By purchasing the MalleoTrain® S you have selected a product that meets very high medical and quality standards.

### Purpose

The MalleoTrain® S is an active ankle support. In addition to the support, the strap helps wearers utilize sports taping mechanisms.

A strap helps wearers better feel the movement of their foot. Similar to the effects of sports taping, this optimizes how the body's center of gravity is guided over the foot, helping prevent injuries and accelerating the healing process.

### ⚠ Indications

#### General information

Read all of these instructions carefully because they contain important information about the use, indications, contraindications, application, care and disposal of the MalleoTrain® S. Keep these instructions for use. You may want to read them again. If you have any questions, contact your treating physician, your specialist store or our Technical Service department.

#### Information about wearing the product

Do not use with other products, such as compression stockings, unless you discuss this in advance with your doctor. Do not let the product come in contact with substances containing fats or acids, ointments or lotions.

If your symptoms worsen or you notice any unusual changes, please contact your doctor immediately.

The manufacturer is exempt from product liability if the product is used improperly.

This product is most effective when the wearer is physically active.

Take the MalleoTrain® S off if resting for long periods (e.g., sleeping, sitting for a long time).

Do not make any changes to this product. If this restriction is not observed, the product's performance may be compromised so that product liability is excluded.

### Indications

- Ligament insufficiency
- Post-operative rehabilitation
- Mild sprains
- Prevention of recurrence, especially when playing sports

### Parts of the body this product is used for

As stated under Indications (ankle).

## Risks of using this product

To date, there are no known side effects that impact the entire body. The product must be put on/used properly. Do not put the MalleoTrain® S on too tightly or it could create localized pressure symptoms. In rare cases constriction of the blood vessels and nerves is possible.

## Contraindications

There are no known medically significant hypersensitivities. With the following clinical presentations you should only put on and wear this type of medical after consulting with your doctor:

- Skin problems/injuries to the part of the body where the support is worn, especially if there are signs of inflammation. Also, raised scars with swelling, redness and overheating
- Sensation/movement disorders in the legs/feet, e.g., associated with diabetes
- Lymphatic drainage problems, including undiagnosed soft tissue swelling in a part of the body other than where the support is worn.

## Pulling/putting on and removing the MalleoTrain® S

- A – support  
B – strap

### Putting the product on

Pull the support on so that the spot where the strap attaches to the support is located on the inside of the foot. The direction the strap needs to be wound around the foot can be found on the end of the strap that attaches to the support. The strap is wrapped around your foot and back of the ankle in a figure 8. To do this, please do the following:

When wrapping the strap, please make sure there are no wrinkles. Bring the strap under the sole of your foot toward the outside of your foot. By pulling the strap you can stabilize the foot and ankle comfortably. Bring the strap diagonally across the top of the foot and back around ankle. Bring the strap diagonally forward, to the inside of the foot. Bring the strap under the sole of your foot toward the outside of your foot to the top of your foot. Now fasten the end of the strap to section of the strap that is underneath it.

Alternatively you can use the designs on the support to guide you. Blue dots on the titanium-colored knit fabric indicate where the strap is supposed to go. When wrapping the strap around your foot, follow the dots knitted into the fabric, starting with one dot, going to two dots and then to three dots.

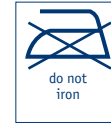
## Care instructions

**Note: Never expose the MalleoTrain® S to direct heat (e.g., heater, sunlight, keep it in your car, etc.).** This could damage the materials, possibly making the SacroLoc® less effective.

Close the Velcro fastener to prevent damage from occurring.

6 Wash the support separately in a lingerie bag with a mild laundry detergent at 30°C.

Do not run the spin cycle on your washing machine at speeds higher than 500 rpm.

Wash the MalleoTrain® S regularly so that the fabric retains its compressive properties.

## Maintenance instructions

If properly handled and cared for, this product is maintenance free.

## Setup and assembly instructions

The MalleoTrain® S comes ready to use.

## Technical specifications/parameters, accessories

The MalleoTrain® S is an ankle support with an attached strap. The strap cannot be removed. There are no accessories for this product.

## Information about re-use

The MalleoTrain® S is intended for your use only, so do not give the MalleoTrain® S to anyone else.

## Warranty

This product is guaranteed in accordance with statutory regulations and Bauerfeind AG's general terms and conditions (GTC) if all of the aforementioned points are adhered to.

## Disposal

You can dispose of this product with your household garbage.

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