medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
A-6020 Innsbruck
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Voortstraat 79
3580 Beringen / Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

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Lumbar support with Clima Comfort

Dear Patient

We are pleased that you have decided to use the high-quality product from medi and thank you for the trust you have shown in us.

Please read these instructions before using the support for the first time. If you apply the support correctly, it will develop its complete medical efficacy and it will be comfortable to wear – so that you will feel better soon.

Areas of use

- Non-specific pain in the area of the lumbar spine
- Wear-related pain in the area of the lumbar spine
- Irritation of the sacro-iliac joint
- Excessive stress on the border between the lumbar spine and the pelvis
- Weak or imbalanced musculature in the trunk (back or stomach)
- Loosening of the pubic symphysis

Side-Effects / Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. If unexpected pain, swelling or numbness occurs while wearing the support, remove it and seek medical advice. Particular care is required where the following conditions are present:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as unclear swellings of soft tissues, apart from the area of use.

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of Action/Comment on Wear

With the combination of the compressive knitted fabric and the integrated back rods, Lumbamed basic stabilises the area of the lumbar spine and the border to the pelvis, without impairing muscular strength. The static forces in the vertebral column are improved and stress on the affected segments is reduced. This alleviates pain.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice. The support is suitable for use in water. After use, please rinse out the product thoroughly.

Putting on/Taking off

- First slip the left hand into the internal wrist strap of the left seal side. Guide the bandage round the body and then slip the right hand into the external hand strap of the right seal side (Figs 1+ 2).

- Pull both sides of the seal in the middle in such a way that they are attached to each other by Velcro pads as evenly as possible. The bandage is sitting possibly when the lower edge of the seal is lying just over the edge (Figs 3 + 4)

Please always wear the Lumbamed basic on the skin and under your clothing. This maximises the Clima Comfort Effect. Please take care that trained staff instruct you when putting on the support for the first time.

Instruction for Washing / Care Tips

Velcro tabs must be closed for washing. Wash the product separately with a woollen or mild detergent up to 30° with gentle wash action or by hand. Rinse thoroughly and hang up to dry. Do not tumble dry, on a radiator or directly in the sun. Do not remove spots with benzine or chemical detergents. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

Note to Storage

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Composition of the Material

Polyamide, elastan

Statutory Defects Liability/

Contractual Guarantee

Consistent with legal guidelines.

Important Notes

It is recommended that the support should only be used for single patient. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law.

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