medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
A-6020 Innsbruck
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Voortstraat 79
3580 Beringen / Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Danmark
Nordre Industrivej 2
6270 Tønder/Danmark
T +45 74 72 01 33
F +45 74 72 01 63
info@nygaardseftf.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona/Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL/England
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

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Manumed[®] active

Dear Patient,

Thank you for choosing a quality product from medi. Before you apply the bandage for the first time, please read the instructions for use. This will help to ensure that you receive the maximum benefit from this quality product.

Areas of use

- Wrist injuries, e.g. sprains, distortions
- Wear of the wrist joint (arthrosis)
- Feeling of wrist instability
- Tendon sheath inflammation
- Pain in the wrist area (e.g. rheumatic pain)

Contraindications

There are no known side effects. Please consult with your doctor if you suffer from any of the following:

- Diseases or injuries to the skin in the area in which the product is used, particularly if there are signs of inflammation (overheating, swelling or reddening)
- Changes in the sensitivity and perfusion of the lower arm and hand
- Changes in lymph drainage and swellings in the soft tissue away from the area of use and without a clear explanation
- Compression syndromes (pressure irritation to the nerves in the arms)

Bandages or orthopaedic devices can in principle cause local compression symptoms if they are applied too tightly. In rare cases, this may cause constriction of blood vessels or nerves.

Mode of action / Product properties

The Manumed active is knitted to the form of the wrist and is fitted with a maleable rod on the under side. This stabilises the wrist and is held in place by a Velcro band. The stabilising effect of the bandage restricts the movements of the wrist, thus reducing stress on the muscles of the lower arm and the points of attachment of the tendons in the elbow. This helps to reduce irritation. The belt band has an opening to prevent excessive pressure on the external knuckle. The compression of the knit aids rapid reduction of swelling and blood effusion.

Wearing the bandage

During extended periods of rest, the Manumed active wrist bandage should be taken off (e.g. during sleep). If there are unexpected excessive changes, e.g. exacerbation of the symptoms, the product must be taken off at once and the medical advice sought.

Moulding the wrist rod

The wrist rod can be individually moulded at temperatures of about 200 °C.

Instructions for application

Before applying the support, open the belt band.

1. Turn the lower end of the support inside out. The rod is then pushed into the special pocket on the inside of the support. Take care that the orientation of the rod in the pocket is correct.
2. Now pull the support over the hand. The “m” must be visible on the back of the hand.
3. Close the belt buckle in such a way that the opening is positioned over the external knuckle.

To remove the support, open the belt band. Grasp the support from the palm. Put pressure on the internal pocket and rod and pull the support from the hand.

In principle, this product must be used under medical supervision. When it is applied for the first time, this must be under the instructions of trained and specialised personnel, such as an orthopaedic technician. If the support is individually fitted by a qualified person, this will ensure that it is effective and comfortable.

If it is to be used with other products, this must be discussed with the prescribing doctor.

Care instructions

Velcro fasteners should be closed for washing. Remove the aluminium splint before washing. Wash the product at 30 °C on a delicates setting or by hand using a mild detergent such as woolwash. Rinse the product thoroughly and hang up to dry. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Fabric softeners, grease, oil, lotions and ointments can affect the bandaging material and inhibit the Clima Comfort effect. Soap residues can cause skin irritations and wearing of the material.

Your medi team

wishes you a quick return to full fitness.

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